


Smile Line ☺
made in switzerland

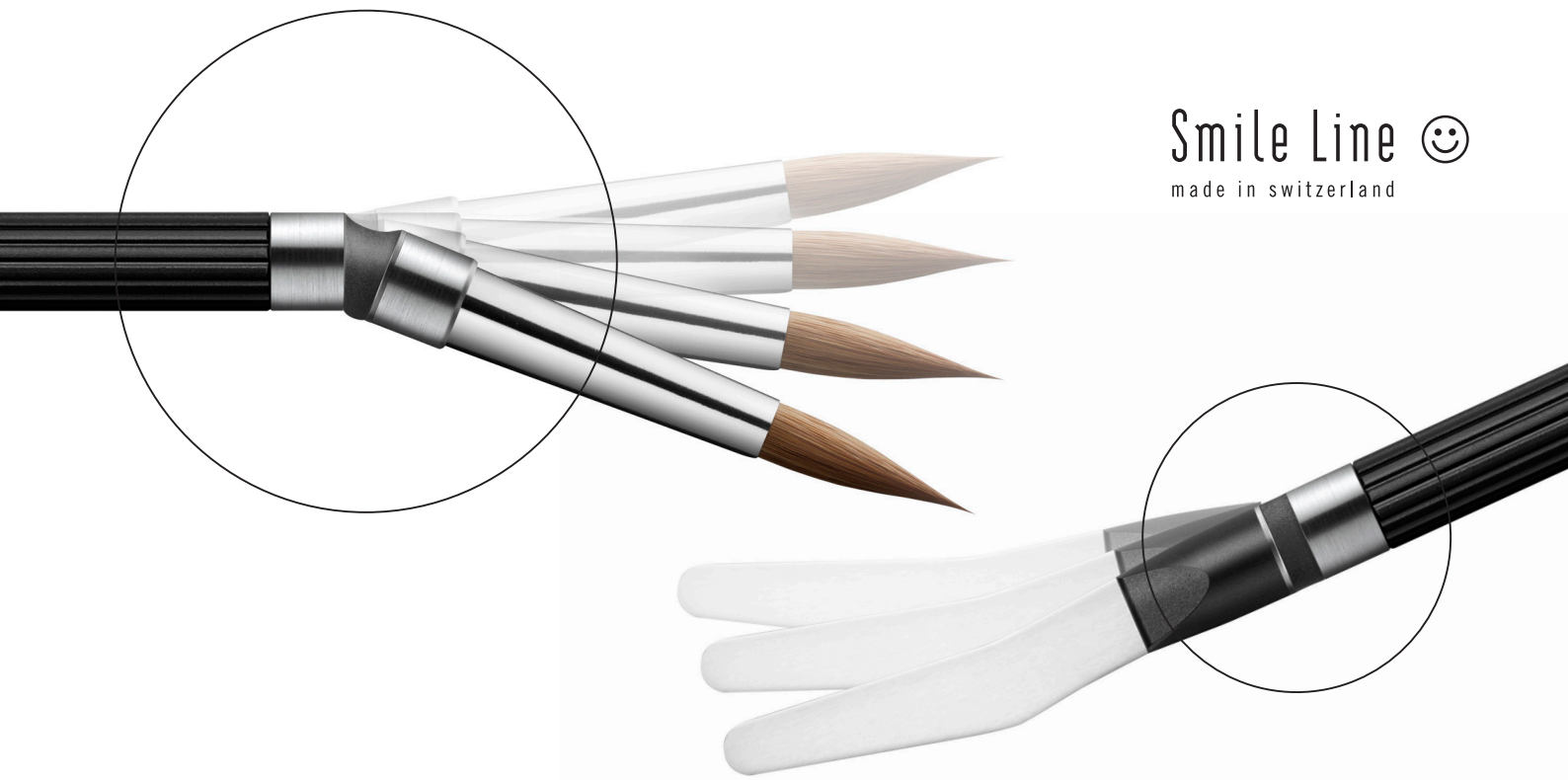

FLEXMOUNT

simple – as the great ideas

Smile Line introduces «**FlexMount**», a flexible rubber connector for combining the handle and the tool. This new concept is adapted to Instrument by Smile Line, more specifically to our ZR mixing spatula and our Porcelain Brushes.



„FlexMount“ with your ZR spatula –



Zirconia is a high-tech material that doesn't have natural elasticity and is fragile for our application. **«FlexMount»** works as a shock absorber and allows for maximum comfort and feeling during mixing of porcelain or stains.



„FlexMount“ with your Porcelain Build-Up Brush –

The **«FlexMount»** allows you to apply a very gentle snap to the brush tip.

Challenge **①** – we need the brush tip to have the right moisture for a precise and comfortable placement of the porcelain. A gentle snap «accelerates» the required moisture from the brush belly towards the point.

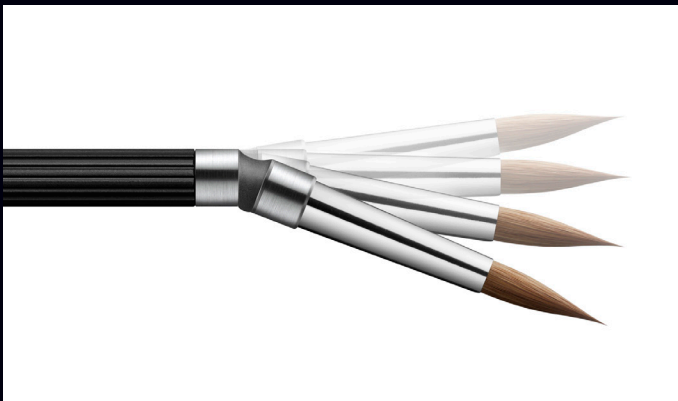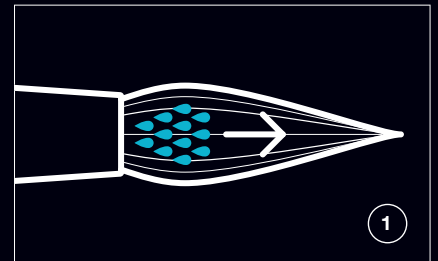

Challenge **②** – we need to have an extremely precise, sharp and stable brush tip. This is mainly provided by the quality of the hair and the way the tip is constructed. A gentle snap right before picking up the porcelain helps the hair at the tip to get back into a sharp point.

Use & Advice

The idea is not to snap the brush immediately when taking it out of the water cup. This creates water spattering around the working space (not very comfortable) plus this method doesn't allow the best humidity control in the brush tip. Smile Line's proposal is to first «stop» the brush on the H₂O sponge for extracting the excess of moisture in the brush tip.

Then, only during porcelain build up we are gently snapping, for example between two pick-ups of porcelain. Utilizing this method, your brush tip is automatically receiving the ideal moisture plus is returning to its sharpest shape.



«FlexMount» is compatible with the following products:

- Smile Line's ZR spatula
- Smile Line's Kolinsky brush tips #8-R, #8-LT, #6 *
- Smile Line's N.era 8 brush tip *

Product detail




* **Note:** Brush tips for «FlexMount» are a completely new and very high-end production. Also, they have a shorter ferrule that allows for more elasticity.


How and what should I order?

Smile Line always provide Instrument parts separate as the majority of our items are modular. If you already have a spare Smile Line instrument handle, we suggest you order a module «flexible connector & tips» so you are able to benefit from the shorter ferrule.

Another favorable option is to ask for your personal «**FlexMount – 2 instruments set**» which contains one brush and one zirconia spatula. You can customize your set to include your favourite brush size and preferred handle material. The handles are available in the new elegant carbon fiber or the traditional anodized aluminium. Ask your dealer !

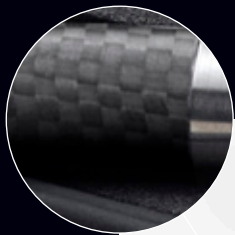

CARBON FIBER

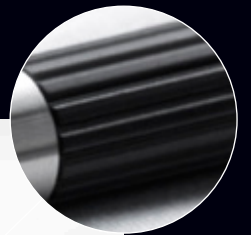

ANODIZED ALUMINUM



#FL-14150-CA6 **Kolinsky #6 + ZR**



#FL-14150-6 **Kolinsky #6 + ZR**



#FL-14150-CA8K **Kolinsky #8R + ZR**



#FL-14150-8K **Kolinsky #8R + ZR**



#FL-14150-CA8N **N.era 8 + ZR**



#FL-14150-8N **N.era 8 + ZR**



FLEXMOUNT

A SUPERIOR QUALITY, A UNIQUE STYLE, AN EXCLUSIVE FEEL...
THERE ARE PLENTY OF REASONS TO PURCHASE AN OBJECT
FROM SMILE LINE.

Smile Line ☺
made in switzerland